



BIORAD® PROTECTIVE

Washing And
Disinfection Machines



FORMS OF PACKAGE

Description	Package
1 L PE bottle	20×1 L
5 L PE drum	4×5 L
10 L PE drum	1 pc.

CHEMICAL COMPOSITION:

Medical paraffin, Benzalkonium Chloride, Mono Ethylene Glycol and Anti-corrosive agents

PRODUCT SPECIFICATIONS:

PROTECTIVE provides the preventative maintenance and disinfection of the instruments used in all medical and dental fields. It prevents the devices from getting worn and allows their joints to function comfortably with its medical grade white oil content. This oil film covers the surface effectively and prevents the contact with air and moisture.

USE:

The solution is ready-to-use. Sufficient amount of ready-to-use Biorad®Protective emulsion is discharged inside washer-disinfectors or immersion baths. All the instruments to be protected are immersed to remain completely inside the emulsion. Medical, surgical and dental instruments are allowed to dry after being taken from washer-disinfectors or immersion baths. The instruments can be delivered directly to sterilization after this procedure, if necessary. It is sufficient for the treated instruments to be disinfected with any disinfectant solution when they will be used again. The solution is recommended to be replaced when visible contamination occurs.

AREAS OF USE:

It is compatible with metals, plastics, rubber, glass, wood, porcelain and stainless steel materials.