



BIORAD® STRONG A PLUS

Washing And
Disinfection Machines



FORMS OF PACKAGE

Description	Package
1 L PE drum	20×1 L
5 L PE drum	4×5 L

CHEMICAL COMPOSITION:

Phosphoric acid, Benzalkonium chloride, Anti-corrosive agents

USAGE CHARACTERISTICS:

It is a neutralizing solution which contains liquid and organic acid and used for all washerdisinfectors. It is an effective product in the neutralization of remaining alkaline residues and removal of the potential lime scales inside the washing machines in main washing procedures. pH=1-3.

USAGE AREAS AND PURPOSE:

Strong A PLUS rust and corrosion inhibitor removes the stains, corrosion and rust on medical devices. It can be used in several fields such as surgery, anesthetics, laboratory equipment, dentistry and etc.

USE:

BIORAD STRONG A PLUS Rust and corrosion inhibitor is used by dilution with water in any quality (e.g. at 1% concentration; 10 mL of solution in 1 L of water) depending on the rust state of the surface to be cleaned. The product that is diluted in appropriate ratio is filled inside a container, the rusted instruments are immersed into the container so as all the surfaces will contact with the solution, they are taken out of the solution after waiting for minimum 5 and maximum 10 minutes and then rinsed with plenty of water. The action time in ultrasonic devices is 5 minutes. The usage period of the prepared solution is 24 hour; replace the solution earlier in case of increase in contamination