



# BIORAD®

## CLEAN+AK SPRAY

Surface And Floor  
Disinfectants



### FORMS OF PACKAGE

Açıklama	Ambalaj
1 L PE bottle	20×1 L
5 L PE drum	4×5 L

### CHEMICAL COMPOSITION:

Didecyldimethylammoniumchloride , Propane-2-ol, Ethanol, Corrosion inhibitor and excipients

### PRODUCT SPECIFICATIONS:

It is an alcohol based, anti-corrosive product which has a pleasant odor and used for the quick disinfection of medical device surfaces. It is dermatologically tested. It is used as an aid in the quick disinfection of all medical devices, manual instruments, equipment and their surfaces which are alcohol-resistant and suitable for moistening according to the medical devices regulation.

### AREAS OF USE:

It is used for the cleaning and disinfection of all horizontal and vertical surfaces and especially PVC epoxy areas, floors, walls, ultrasonography probes, incubators, aerators and micromotor tips used in dentistry, stethoscopes, cabinets, doors, examination, disinfection requiring medical devices and equipment in ENT units as well as operating rooms, intense care rooms, sickbed edges, iv holders, microscopes, areas contacting with hands and skin and tools contacting with patients' skin in the hospitals that require quick disinfection.

### USE:

Clean+Ak Spray, is a ready-to-use spray disinfectant suitable for use on all horizontal and vertical surfaces. It is sprayed from a distance of 30cm-40cm . It takes 30 seconds for the effect to appear. It is wiped with a clean cloth and allowed to dry.

### ANTI-MICROBIAL CHARACTERISTICS:

Bactericide, Fungicide, Virucide, Antimycobacterial, Tuberculocide