



# BIORAD®

## READY HP SPRAY

Surface And Floor  
Disinfectants



### FORMS OF PACKAGE

Açıklama	Ambalaj
1 L PE bottle	20×1 L
5 L PE drum	4×5 L

#### CHEMICAL COMPOSITION:

Hydrogen peroxide, deionized water

#### PRODUCT SPECIFICATIONS:

It is a hydrogen peroxide based, anti-corrosive product which has a pleasant odor and used for the quick disinfection of medical device surfaces. Ready Hp Spray is a ready-to-use and highly efficient medical device disinfectant. Its formulation is based on hydrogen peroxide. It is a disinfectant which does not contain aldehyde, alcohol and phosphate and is anti-corrosive and material-resistant. It is dermatologically tested.

#### AREAS OF USE:

It is used for the cleaning and disinfection of all horizontal and vertical surfaces and especially PVC poxy areas, floors, walls, ultrasonography probes, incubators, aerators and micromotor tips used in dentistry, stethoscopes, cabinets, doors, examination, disinfection requiring medical devices and equipment in ENT units as well as operating rooms, intense care rooms, sickbed edges, iv holders, microscopes, areas contacting with hands and skin and tools contacting with patients' skin in the hospitals that require quick disinfection.

#### USE:

Ready Hp Spray, is a ready-to-use spray disinfectant suitable for use on all horizontal and vertical surfaces. It is sprayed from a distance of 30cm-40cm . It takes 30 seconds for the effect to appear. It is wiped with a clean cloth and allowed to dry.

#### ANTI-MICROBIAL CHARACTERISTICS:

Bactericide, Fungicide, Virucide, Antimycobacterial, Tuberculocide

[www.famed.com.tr](http://www.famed.com.tr)