



BIORAD®

READY SPRAY

Surface And Floor
Disinfectants



FORMS OF PACKAGE

Açıklama	Ambalaj
1 L PE bottle	20×1 L
5 L PE drum	4×5 L

CHEMICAL COMPOSITION:

Benzalkonium chloride, Didecyldimethylammoniumchloride and excipients

PRODUCT SPECIFICATIONS:

Ready Spray is ready-to-use and has an extended antimicrobial action spectrum with the quaternary ammonium compounds in its composition. It does not contain aldehyde, phosphate and alcohol. It is recommended to use on surfaces with metal parts since it contains corrosion inhibitor. Ready Spray is compatible with glass, ceramic, silicone, plastic (including Plexiglas), wood and stainless steel materials.

AREAS OF USE:

It is used for the cleaning and disinfection of all horizontal and vertical surfaces and especially incubators, PVC epoxy areas, floors, walls, ultrasonography probes, incubators, aerators and micromotor tips used in dentistry, disinfection requiring medical devices and equipment in ENT units as well as operating rooms, intense care rooms, in the hospitals that require quick disinfection.

USE:

Ready Spray, is a ready-to-use spray disinfectant suitable for use on all horizontal and vertical surfaces. It is sprayed from a distance of 30cm-40cm . It takes 30 seconds for the effect to appear. It is wiped with a clean cloth and allowed to dry.

ANTI-MICROBIAL CHARACTERISTICS:

Bactericide, Fungicide, Virucide, Antimycobacterial, Tuberculocide